



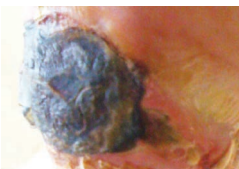


# APPLICATION GUIDE

## When to Use Which MediHoney® Dressing

TYPE OF WOUND	OBJECTIVE	EXUDATE	MEDIHONEY DRESSING (PRIMARY DRESSING)	XTRASORB DRESSING (SECONDARY DRESSING)
Necrotic 	Debride	Dry	HCS	N/A
		Light	Wound Gel • Medical Honey	Foam • HCS
		Moderate	Apinate	Classic
Infected/ Odourous 	Reduce Bacterial Load/Eliminate Odour	Light to Moderate	HCS • Medical Honey Tulle • Wound Gel	Foam • HCS
		Heavy	Apinate	Classic
Sloughy 	Remove Slough	Light to Moderate	HCS • Medical Honey Tulle • Wound Gel	Foam • HCS
		Heavy	Apinate	Classic
Granulating 	Promote Granulation	Light to Moderate	Gel Sheet • HCS Tulle • Wound Gel	Foam • HCS
Epithelialising 	Maintain Moist Environment	Dry to Moderate	HCS	N/A



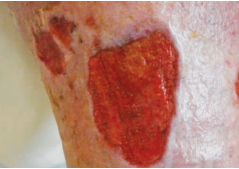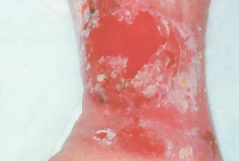


## MediHoney® Antibacterial Dressing

	Product Code	Description	Packaging Unit/Case
<b>Wound Gel</b>	DUP391	10 g	20/box, 6 boxes/case
	DUP395	20 g	5/box, 36 boxes/case
<b>Medical Honey</b>	DUP398	20 g	5/box, 36 boxes/case
	DUP405	50 g	1/box, 24 boxes/case
<b>HCS Non-Adhesive</b>	DUP780	6cmx6cm (2.4inx2.4in)	10/box, 5 boxes/case
	DUP781	11cmx11cm (4.3inx4.3in)	10/box, 5 boxes/case
	DUP784	20cmx20cm (8inx8in)	5/box, 4 boxes/case
	DUP785	20cmx30cm (8inx12in)	2/box, 5 boxes/case
	DUP782	7.2cmx7.2cm (2.8inx2.8in), 11cmx11cm (4.3inx4.3in), with adhesive border	10/box, 5 boxes/case
<b>Adhesive</b>	DUP783	11.5cmx11.5cm (4.5inx4.5in), 15cmx15cm (6inx6in) with adhesive border	10/box, 5 boxes/case
	DUP787	4.5cmx16.5cm (1.75inx6.5in) 7.5cmx20cm (3inx8in), with adhesive border	10/box, 5 boxes/case
<b>Apinate Dressing Rope</b>	DUP793	Rope, 1.9cmx30cm (3/4inx12in)	5/box, 4 boxes/case
<b>Dressing</b>	DUP794	5cmx5cm (2inx2in)	10/box, 10 boxes/case
	DUP795	10cmx10cm (4inx4in)	5/box, 5 boxes/case
	DUP796	3-ply, 10cmx10cm (4inx4in), unfolds to 30cmx10cm (12inx4in)	5/box, 20 boxes/case
<b>Tulle Dressing</b>	DUP797	3-ply, 5cmx5cm (2inx2in), unfolds to 15cmx5cm (6inx2in)	5/box, 20 boxes/case
	DUP798	5cmx5cm (2inx2in)	10/box, 10 boxes/case
<b>Gel Sheet</b>	DUP799	10cmx10cm (4inx4in)	10/box, 10 boxes/case

## Xtrasorb® Super Absorbent Dressing

	Product Code	Description	Packaging Unit/Case
<b>HCS Non-Adhesive</b>	DUP86322	6cmx6cm (2.3inx2.3in)	10/box, 4 boxes/case
	DUP86344	11cmx11cm (4.5inx4.5in)	10/box, 4 boxes/case
	DUP86388	20cmx20cm (8inx8in)	5/box, 8 boxes/case
<b>Adhesive</b>	DUP86433	8cmx8cm (3inx3in)	10/box, 4 boxes/case
	DUP86466	15cmx15cm (6inx6in)	10/box, 4 boxes/case
<b>Foam</b>	DUP86122	5cmx5cm (2inx2in)	10/box, 4 boxes/case
	DUP86144	10cmx12cm (4inx4.75in)	10/box, 4 boxes/case
	DUP86188	20cmx20cm (8inx8in)	5/box, 8 boxes/case
<b>Adhesive</b>	DUP86233	8cmx8cm (3inx3in)	10/box, 4 boxes/case
	DUP86244	11.5cmx11.5cm (4.5inx4.5in)	10/box, 4 boxes/case
	DUP86266	15cmx15cm (6inx6in)	10/box, 4 boxes/case
<b>Classic Non-Adhesive</b>	DUP89533	7.6cmx7.6cm (3inx3in)	10/box, 10 boxes/case
	DUP89544	8.5cmx8.9cm (3.375inx3.5in)	10/box, 10 boxes/case
	DUP89545	10.1cmx12.7cm (4inx5in)	10/box, 10 boxes/case
	DUP89569	15.2cmx22.9cm (6inx9in)	10/box, 5 boxes/case

Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region.

- Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.
- Consult product labels and inserts for any indication, contraindications, hazards, warnings, precautions, and instructions for use.

### For more information please contact:

#### Integra Canada ULC

2590 Bristol Circle, Unit 1  
Oakville, Ontario L6H 6Z7  
Phone: 1-800-263-6693 (Toll Free) Fax: 905-632-7938  
IntegraCanadaCustSvc@integrallife.com



### Manufacturer:



Derma Sciences, Inc.  
104 Shorting Road  
Toronto, Ontario M1S 3S4 ■ Canada



Integra LifeSciences Services (France)  
Immeuble Séquoia 2  
97 Allée Alexandre Borodine  
Parc Technologique de la Porte des Alpes  
69800 Saint Priest – France